



## reTyre TRACTION



Easy and quick to install



Don't need to adjust the wheel locks



Roll into them and go



Panthera has partnered with reTyre to develop reTyre Traction Skins for wheelchair tires. Designed for wheelchair driving, the tire skins provide excellent traction through snow, while at the same time making it easier for the user to propel forward and brake. Turning corners and moving through snow, slush and rain are so much easier with this rugged accessory.

panthera

# 30-second installation



Attach the clip



Align with tire



Roll over



Fasten end



Best of all, the reTyre Traction uses Velcro attachments to fit over your own wheelchair tires like a second skin, and works with your brakes without the need to adjust them. Afterwards, they are easily removed at your destination.

reTyre is also excellent for summertime driving on looser surfaces such as grass and gravel, and gives you extra puncture protection. Because you can remove them before going inside, you can avoid bringing gravel and dirt into your house. The set of tires comes with a handy storage bag.

Currently, **the tire skins are available for 24" drive wheels, suitable for all 24" x1 tires** with a circumference of 1885 mm (+/-10mm) reTyre Traction is adapted in size so that after assembly there is a gap of 0-22 mm, which account for the variations that may occur due to differences in manufacture or wear. This gap does not affect function or driving experience.

Triumph Mobility  
151 Bentley Ave, Unit 4, Ottawa, ON, K2E 6T7  
Tel: +1 (855) 546 0711 Fax: +1 (866) 225 3059  
Email: [info@triumphmobility.com](mailto:info@triumphmobility.com)  
[www.triumphmobility.com](http://www.triumphmobility.com)



Exclusive Distributor in North America



TriumphMobility