


Brake handle -
Left

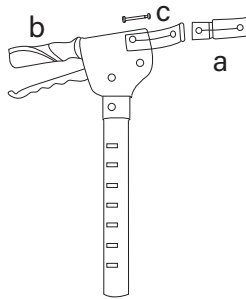

1

Adj. Collar - Left




2

Adj. Collar Sleeve




3

T-Frame - Left

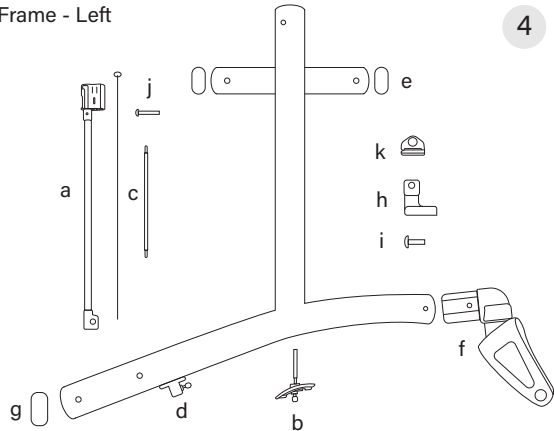


4

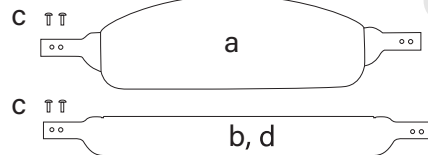


5




6

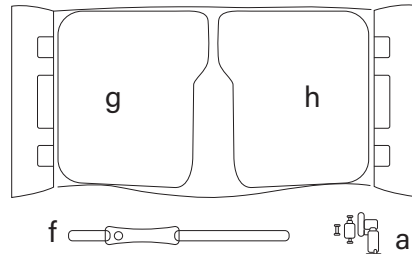

7

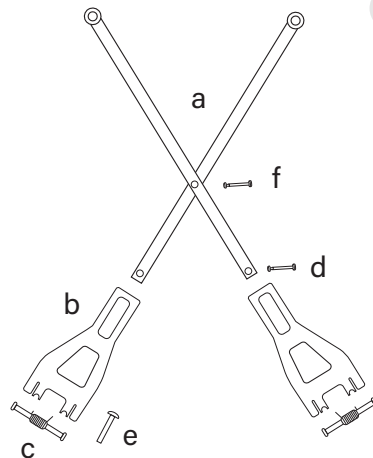

1

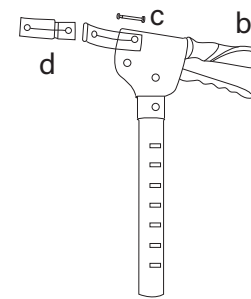

Brake handle -
Right

2




Adj. Collar - Right

3



Adj. Collar Sleeve


4

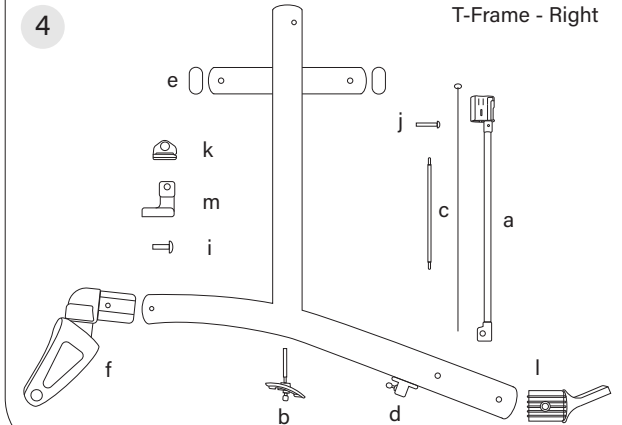

T-Frame - Right


8

Rear wheel - Right

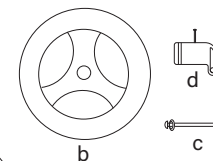

9

Front wheel

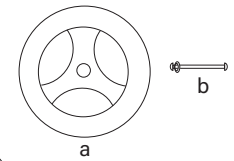

Parts List: Escape Rollator (Escape, Petite)

Click [part number](#) to see part image

1. Brake Handle

	500-1500	Brake Handle 'U' shape rod - Complete - LEFT	EA
	500-1501	Brake Handle 'U' shape rod - Complete (Black) - RIGHT	EA
1a	500-1002	Backrest connector - LEFT	EA
1b	500-1009	Anatomical handles (1 PR)	PR
1c	500-1011	Housing / Backstrap connector screw & nut	EA
1d	500-1005	Backrest connector - RIGHT	EA

2. Adj. Collar

2a	500-1025	Adj Collar complete - LEFT	EA
2a	500-1525	Adj Collar complete (GREY) - LEFT	EA
2b	500-1026	Adj Collar complete - RIGHT	EA
2b	500-1526	Adj Collar complete (GREY) - RIGHT	EA

3. Adj. Collar Sleeve *(common to both sides)*

	500-1027	Locking ring sleeve long	EA
--	--------------------------	--------------------------	----

4. T-frame

4a	500-1430	Lower brake rod ('U' SHAPE) assembly - Low - LEFT	EA
4a	500-1431	Lower brake rod ('U' SHAPE) assembly - Low - RIGHT	EA
4a	500-1432	Lower brake rod ('U' SHAPE) assembly - Super Low - LEFT	EA
4a	500-1433	Lower brake rod ('U' SHAPE) assembly - Super Low - RIGHT	EA
4a	500-1434	Lower brake rod ('U' SHAPE) assembly - Standard - LEFT	EA
4a	500-1435	Lower brake rod ('U' SHAPE) assembly - Standard - RIGHT	EA
4b	500-1440	Brake cable roll out block - Complete (R2)	EA
4c	500-1443	Brake cable complete (R2)	EA
4d	500-1041	Brake cable bracket - Complete	EA
4e	500-1070	Seat Side tube plug (set) - 4pc/SET	EA
4f	500-1448	Front Fork (NEW STYLE) Assembly	EA
4g	500-1062	Bottom tube plug	EA
4h	500-1101	Basket hook - LEFT	EA
4i	500-1102	Basket hook screw	EA
4j	500-1103	Seat to frame screw	EA
4k	500-1104	Red clips for shopping bag (Left & Right)	PR
4l	500-1061	Curb Climber (RED)	EA
4l	500-1063	Curb Climber (BLACK)	EA
4m	500-1100	Basket hook - RIGHT	EA

5. Backstrap

5a	500-1403	Backstrap (Large Padded) - Complete	EA
5b	500-1003	Backstrap - Complete	EA
5c	500-1004	Backstrap screw & Nut - Complete - 2pc/SET	EA
5d	500-1407	Backstrap Escape Petite - Complete	EA

6. Seat

	500-1071	Seat Mechanism - Complete (Assembled)	EA
	500-1072	Seat Mechanism - Petite Complete (Assembled)	EA
6a	500-1080	Frame Lock Complete - Escape	EA
6b	500-1085	Seat Lock Assembly	EA
6f	500-1095	Seat Handle Strap	EA
6g	500-1096	Handle Strap (soft)	EA
6h	500-1596	Seat cushions 1" (L&R) Escape only	EA

7. Cross Bar

7a	500-1110	Cross bar complete - Escape Super Low	EA
7a	500-1112	Cross bar complete - Escape Petite - S. Low	EA
7a	500-1115	Cross bar complete - Escape Low	EA
7a	500-1120	Cross bar complete - Escape Standard	EA
7b	500-1125	Lower cross bar bracket - Super Low	EA
7b	500-1127	Lower cross bar bracket - Escape Petite - S. Low	EA
7b	500-1130	Lower cross bar bracket - Low	EA
7b	500-1135	Lower cross bar bracket - Standard	EA
7c	500-1140	Spring for support arm - 2 pc/SET	EA
7d	500-1142	Cross bar bracket screw - Upper	EA
7e	500-1143	Cross bar bracket screw - Lower	EA
7f	500-1144	Cross bar screw	EA

8. Rear wheel

8a	500-1044	Brake pad complete - LEFT	EA
8b	500-1055	Wheel 200 mm - Complete	EA
8c	500-1059	Rear Axle Bolt	EA
8d	500-1045	Brake pad complete - RIGHT	EA

9. Front wheel

9a	500-1055	Wheel 200 mm - Complete	EA
9b	500-1060	Front Axle Bolt	EA

10. Shopping bag

10a	500-3000	Shopping bag - Escape	EA
	500-3010	Shopping bag - Escape Petite	EA
	500-3100	Shopping bag with zipper - Escape	EA