TRIUMPH  
maverick

Instruction Booklet  
*Livret d'instructions*

# Table of Contents

## *Table des matières*

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# Quick Start Guide

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### **Introduction**

Thank you for your purchase of a Triumph Maverick rollator. The Maverick is designed for an active lifestyle and was created with the lightest material – carbon fibre.

Please ensure that all users review the instructions carefully prior to use and are familiar with safety and operating instructions. Before assembling and using your new Triumph Maverick, please check that no damage to the frame or parts has occurred during shipping.

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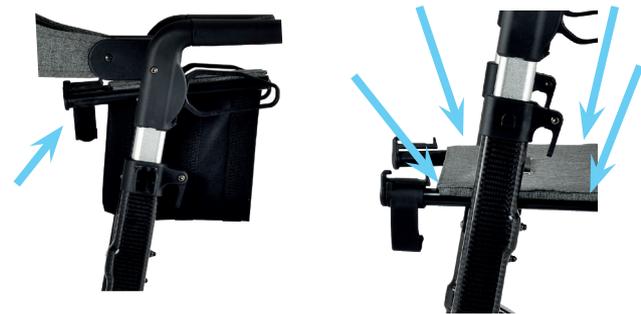
Open the frame lock and pull the brake handles apart. Press the side bars down until you hear a “click” in the fully open position.

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## Installing the backstrap / Installation du dossier souple

Slide the backstrap into the backrest connector. The button should “click” into position.

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Your rollator comes with adjustable handles so you can place your hands at a comfortable height. Lift the handle height adjustment lever, and raise or lower the brake handle to your preferred height. Push down the handle height adjustment lever to lock the brake handles. Make sure your hands are at a comfortable angle with your shoulders back not stooped.

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## Attaching the shopping bag / Installation du sac à provisions

Attach the shopping bag onto the bag hook on the front of the seat.

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Squeeze the brakes to slow down.  
Push down on the brake handles to lock the brakes. Pull up on the brake handles to unlock the brakes.

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## Adjustment of brakes / Ajustement des freins

If the brake is too loose, loosen the locking nut, and turn the adjustment screw counter-clockwise. If the brake is too tight, loosen the locking nut, and turn the adjustment screw clockwise.

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Stand up as straight as possible and walk with the rollator close to your body. Keep your feet centred. Do not push the rollator away from your body, as this will cause you to bend your back, and could potentially cause a fall.

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To sit on the rollator, ensure that the brakes are in the LOCKED position, and the rollator does not move. Using the handles for stability, carefully turn around. Gently back up until the back of your legs are touching the edge of the seat, and sit down. Ensure both feet make contact with the ground while seated.

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## Curb climber / Monte trottoir

We recommend you use the Maverick's built-in curb climber. This is designed to save you lifting the rollator over obstacles such as thresholds and curbs. Just step down on the curb climber and with a gentle pull back on the handles, the rollator will raise its front wheels to easily move over obstacles.

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## Cane holder / Porte-canne

The rollator comes with a cane holder which consists of two parts, an upper hook and a lower foldable base.

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## Cup holder / Porte-gobelet

The rollator comes with a cup holder which is ideal for transporting drinks.

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## Folding and Storage / Pliage et entreposage

To fold the rollator, pull up on the centre seat strap. The rollator can now be folded and locked using the frame lock.

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## Maintenance / Entretien

- ✓ Regular maintenance will preserve the life and safety of your Maverick. The following should be checked at regular intervals.

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- ✓ To adjust brake tension: If the brake handle is too tight, loosen the locking nut, and turn the adjustment screw clockwise. Re-tighten the locking nut. If the brake handle is too loose, loosen the locking nut, and turn the adjustment screw counterclockwise. Re-tighten the locking nut.

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## Warranty / Garantie

Triumph Mobility covers the Maverick Rollator with a lifetime warranty on the frame, and a five-year warranty on the brake handles against defects in material and workmanship. All other components are warrantied for two (2) years.

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This warranty is valid for the original purchaser only and is not transferrable. Triumph Mobility will not warranty any parts that have been damaged due to improper installation, modification, and/or misuse. For warranty service, please return your rollator to your local dealer. Service charges are of the discretion of the dealer and are not the responsibility of Triumph Mobility.

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**Why register?** It only takes a few minutes and by registering you gain:

- Access to theft and vandalism insurance
  - The latest news about Triumph products
  - An extended parts warranty to two years
- When registering, please include the serial number. The serial number can be located on the left side of frame.

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## Theft & Vandalism Insurance / Assurance vol et vandalisme

Our clients know that they can depend on our Triumph rollators. They can rely on us to replace it if it stolen or vandalized. Every Triumph Maverick comes with automatic insurance for your peace of mind.

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For a \$100 deductible (plus shipping and applicable taxes), we will take care of getting a replacement to your doorstep.

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This insurance covers the first two years that you own your Triumph Maverick. (Like most guarantees, it

is not transferrable if the rollator changes ownership).

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To make a claim, we ask our clients to visit:

**[www.triumphmobility.com/product-insurance](http://www.triumphmobility.com/product-insurance)** to provide us with a copy of the relevant police report, the serial number of their Maverick, and proof of purchase. We'll do the rest. Let us help you get mobile again!

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Read instructions carefully before use  
Maverick is designed for indoor/outdoor use  
Instruction manual must be given to the user

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1.855.546.0711

File Name: Maverickusermanual\_R1.

This manual is subject to updates and changes without notice.

Please check [www.triumphmobility.com](http://www.triumphmobility.com) for most current version of this document.

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©2023 Triumph Mobility Inc. All rights reserved. Triumph™ is the trademark of Triumph Mobility Inc.

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