Triumph Essentials Shopping Rollator

Item#: 700-5000

Instruction Booklet
Livret d'instructions

TRIUMPH
essentials

Quick Start Guide

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Introduction

Thank you for your purchase of a Triumph Essentials Shopping Rollator. Please ensure that all users review the instructions carefully prior to use and are familiar with safety and operating instructions. Before using your new Triumph Essentials Shopping Rollator, please check that no damage to the frame or parts has occurred during shipping.

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**WEIGHT CAPACITY /
POIDS SUPPORTÉ:
300 LBS (136 KG)**

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For your safety

Please read these important tips to ensure your safety and get the most out of your rollator.

When using your rollator

1. When using the rollator, ensure that the rollator is fully opened and locked in position.
2. Always confirm the handle height adjustment levers are in the closed position.

Care and maintenance

1. Do not use on stairs or escalators.
2. Never use the rollator in a moving vehicle

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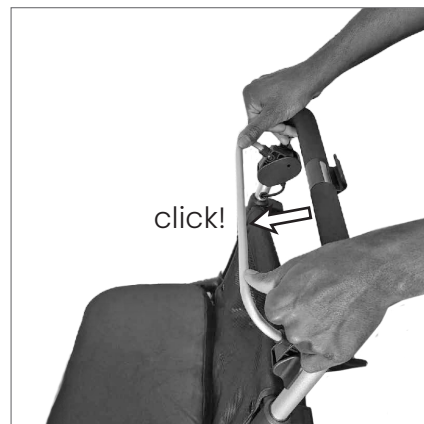
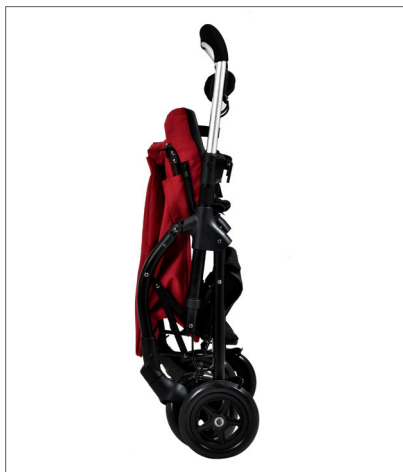
To fold the rollator

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Warranty / Garantie

Triumph covers the Essentials rollator with a two (2) year warranty on the frame. All other components are covered for one (1) year against defects in material and workmanship.

This warranty is valid for the original purchaser only and is not transferable. Triumph Mobility will not warranty any parts that have been damaged due to improper installation, modification, and / or misuse.

For warranty service, please return your rollator to your local dealer . Service charges are of the discretion of the dealer and are not the responsibility of Triumph Mobility.

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Read instructions carefully before use
Triumph Essentials Shopping Rollator is designed for indoor/outdoor use
Instruction manual must be given to the user

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Triumph Mobility
151 Bentley Ave., Unit 4, Nepean, ON, K2E6T7
info@triumphmobility.com
1.855.546.0711

This manual is subject to updates and changes without notice.
Please check www.triumphmobility.com for most current version of this document.

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