

500-1596

Seat Cushions (Set)

INSTALLATION INSTRUCTIONS

Step 1. Remove the original pads



Step 2. Remove the centre seat pad


- Lift the handle strap.
- Cut the seat pad between the two slots.
- Remove the seat pad completely.
- Wipe the plastic seat surface to ensure it is clean and free of dirt.



Tool needed: 

Step 3. Install the New Seat Cushions

- Align the front edge of the seat cushion with the front of the seat.
- Position the left straight edge along the left side of the seat (or the right straight edge along the right side of the seat).


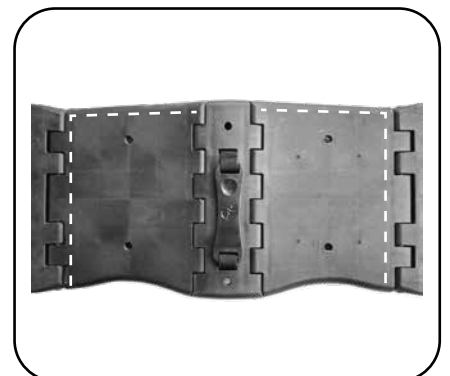


500-1596 Seat Cushions (Set)

INSTALLATION INSTRUCTIONS


Step 3. Install the New Seat Cushions

- Peel off adhesive backing on velcro, and position seat cushions.
- Pushdown on seat cushions to secure velcro.

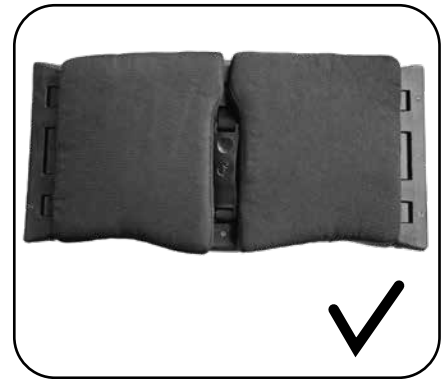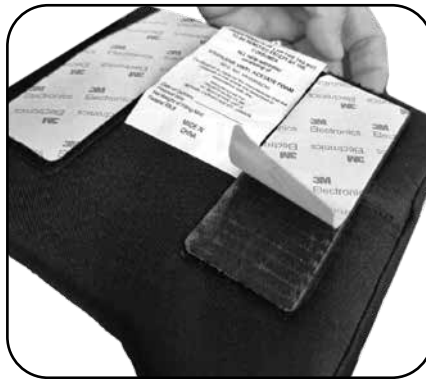

Final Check

Ensure all cushions are securely attached and properly aligned before use.

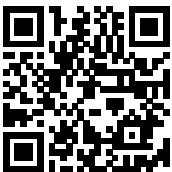

Scan to see the
tutorial video



Contact us

triumphmobility.com
info@triumphmobility.com
1.855.546.0711
151 Bentley Ave., Unit 4, Nepean, ON, K2E6T7